



BEACON

BEACONIC D3

INSTRUCTION FOR USE

| CODE | PRODUCT NAME | PACK SIZE |
|-------|--------------|-----------|
| BD361 | Beaconic D3 | 20 L |

INTENDED USE:

Beaconic D3 is a kind of reliable isotonic diluents which can meet the requirements as follows: Dilute WBC, RBC, PLT HGB. Keep the shape of cells during test process. Offer appropriate background value. Clean WBC and RBC micro-aperture and tubes.

PRINCIPLE:

The cells are counted and sized by the electrical impedance method. This method is based on the measurement of changes in electrical current which are produced by a particle, suspended in a conductive liquid, as it passes through an aperture of known dimensions. An electrode is submerged in the liquid on either side of the aperture in order to create an electrical pathway through it. As each particle passes through the aperture, a transitory change in the resistance between the electrodes is produced. This change produces a measurable electrical pulse. The number of pulses generated is indicative of the number of particles that traversed the aperture. The amplitude of each pulse is essentially proportional to the volume of the particle that produced it. Each pulse is amplified and compared to internal references voltage channels. This channel is delineated by calibrated size discriminators to accept only pulses of certain amplitude. Thus, the pulses are sorted in to various size channels according to their amplitude.

ACTIVE INGREDIENTS

Buffer
Sodium Chloride
Preservative

WARNINGS & PRECAUTIONS

Do not inhale or ingest. In case of inhalation or ingestion, seek medical advice immediately.
Avoid skin contact. In case of skin contact rinse immediately with plenty of water. In case of eye contact rinse immediately with plenty of water & seek medical advice immediately.

STORAGE & USAGE:

The product must be stored between 15-35°C. The shelf life of Beaconic D3 is 2 years from the date of manufacture, if stored at the prescribed temperature range.

Do not use reagent beyond the expiration date printed on the container label. Once opened the product must be used within 60 days.

Do not use reagent once frozen.

You can only proceed to test samples after you have confirmed that the measured blank value is within the required range. Following conditions may produce interference to test result: expired reagent, reagent contaminated by dust, unclean sample dilution, reagent mixed with those of other manufacturer.

SPECIMEN:

Beaconic D3 is intended for use with blood specimens collected either by venipuncture or micro-sampling by skin. Consider all the clinical specimens, controls and calibrators etc. that contain human blood or serum as being potentially infectious, wear lab coats, gloves and safety glasses and follow required laboratory or clinical procedures when handling these materials.

Blood collection and disposal should be performed according to local and national environmental regulations or laboratory's requirements.

Be sure the blood collection clean and contamination-free. All specimens must be properly collected in tubes containing the EDTA (EDTA-K₂·2H₂) anticoagulant used by the laboratory.

Do not shake the sample tube violently. Venous blood can only be stored for 4 hours at room temperature. Manufacturer recommends the blood sample be kept at temperature between 2-8°C for longer storage.

Venous Blood Collection: Collecting whole blood sample through vein-puncture and store in a clean sample tube with EDTA- K₂·2H₂O, which can keep the configuration of WBC, RBC and avoid platelets aggregation. Gently shake the tube 5~10 times to make it well mixed.

Peripheral Blood Collection: Capillary blood is usually collected from fingertip. The volume of sample tube is set to be 20µl.

REAGENT PREPARATION:

This reagent is ready for use and can be applied straight from the container, no special reagent preparation is necessary.



Beacon Diagnostics Pvt. Ltd.
424, New GIDC, Kabilpore,
Navsari-396424
E-mail: sales@beaconindia.com
Doc. No. BEA/24/DB3/HE/IFU-00

Toll free No. 1800-270-1027
web: www.beaconindia.com
Date of issue: 11/05/2023 Rev. No. 00



BEACON

BEACONIC D3

INSTRUCTION FOR USE

REFERENCE RANGE:

WBC: $4 - 10 \times 10^9/L$
RBC: $3.50 - 5.50 \times 10^{12}/L$
HGB: 11 – 15 gm/dL (female)
13 – 17 gm/dL (male)
MCV: 80 – 100 fL
PLT: $100 - 300 \times 10^9/L$

It is recommended that each laboratory establish its own expected reference intervals based up on the laboratory's patient population encountered during daily operation.

DISPOSAL PROCEDURE:

Disposal procedures should meet requirements of applicable local regulations.

PROCEDURE:

Gently shake the tube to well mix the blood sample, the place the sample tube beneath the probe, make sure the probe touches tube bottom slightly. Press run key and remove the sample after hearing beep sound. Process of analysis will take some time, please wait a moment.

MATERIAL REQUIRED BUT NOT PROVIDED

1. Beaconic 3 part Hematology Analyzer
2. LYSE
3. CLEANER
4. EDTA tubes
5. Blood mixer



Symbols used:

| | |
|-------------------------------------|--|
| Manufacturer | |
| In Vitro Diagnostics | |
| Temperature Limitation 15°C to 35°C | |
| Do not hook | |
| Handle with care | |
| Do not stack more than 4 boxes | |
| Keep package Dry | |
| This way up | |



Beacon Diagnostics Pvt. Ltd.
424, New GIDC, Kabilpore,
Navsari-396424
E-mail: sales@beaconindia.com
Doc. No. BEA/24 /DB3/HE/IFU-00

Toll free No. 1800-270-1027
web: www.beaconindia.com
Date of issue: 11/05/2023 Rev. No. 00